



A SHORT LESSON ON LITTERS

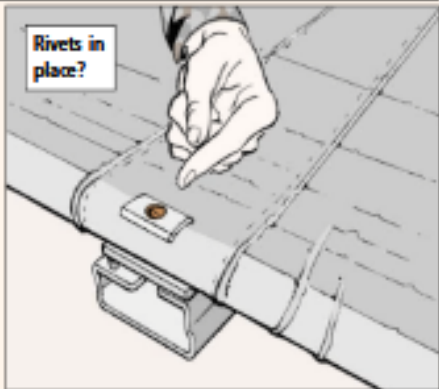


Sometimes even a medical litter, NSN 6530-00-783-7905, needs a little medical attention. Here's a small dose of care:

- Inspect the frame for cracks or wear. Make sure all screws and rivets are tight and in place.

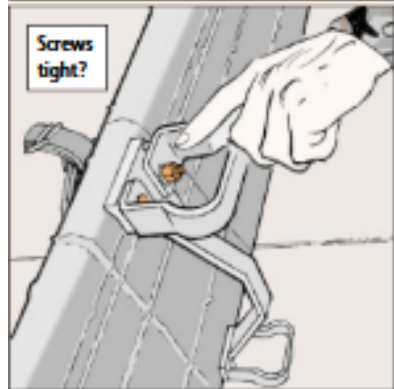

- If the hinge on the spreader bar sticks or shows signs of corrosion, give it a shot of WD-40 or a similar lubricant. That'll loosen rusted parts and protect the metal against moisture. NSN 8030-01-418-9008 gets you an 11-oz aerosol can.

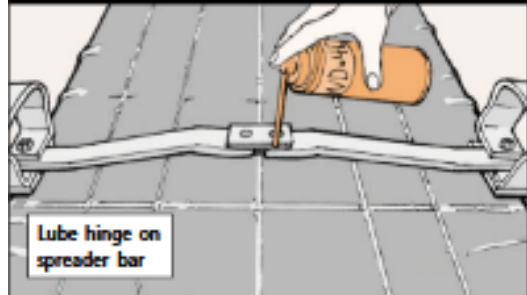

- Look for tears and holes in the nylon cover and repair them while they're still small. You can either tape or patch the cover.




For taping, you'll need a roll of green repair tape. NSN 8315-01-423-6231 gets you a roll 3 inches wide and 50 feet long.

If you plan to patch the cover, the tentage repair kit, NSN 8340-00-262-5767, comes in handy. Cut a patch from the polyester material in the kit and glue it with adhesive, NSN 8040-00-264-3848.

If you find anything on your litter you can't fix or replace, turn it over to direct support.

Replacement Parts

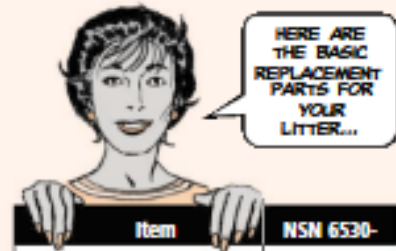

Item	NSN 6530-
Handle	01-247-7157
Spreader bar and stirrups	00-784-3450
Litter pad	00-137-3016
Patient strap, securing	00-784-4205
Securing strap, litter	00-784-4335
Cover, nylon	00-784-1250
Traction appliance	00-926-4732
Backrest	00-299-8353